Opalescence®

tooth whitening systems

EN-Tooth Whitening System

DE-Zahn-Aufhellungs-Systeme

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EN-PATIENT INSTRUCTIONS

Follow your dentist's instructions on how to load gel into your custom bleaching tray. Use no more than 1/3 to 1/2 of syringe.

Brush teeth, then insert tray. Lightly tap tray to adapt tray sides to teeth. **Opalescence 10%, 15%, 20%:** Remove tray after 8-10 hours during the night, or 4-6 hours during the day. **Opalescence 35%:** Remove tray from mouth after 30 minutes.

Remove excess gel with clean finger or soft toothbrush.
Rinse twice; do not swallow rinses.

If significant sensitivity occurs, stop treatment and consult dentist.

Clean tray with soft brush and cool tap water.
Store tray in case provided.

Nonvital Bleaching: Irrigate opening as instructed by dentist. Place gel into lingual access and tray.

Bleaching Precautions:

1. Do not use household bleach to whiten teeth.
2. Pregnant women should not bleach.
3. Keep gel out of heat/sunlight. Do not freeze.
4. Tooth fillings and crowns do not respond to tooth bleaching.
5. Foods and juices high in acid may cause sensitivity.
6. If you have any questions regarding the appropriate use of this product, please consult your dentist as soon as possible.
7. Do not use tobacco products or eat while bleaching.

General Precautions:

1. Teeth are naturally darker along the gumline. These areas may require more time to lighten than the rest of the tooth surface and usually remain slightly darker.
2. A small percentage of patients experience sensitivity with bleaching. Should this occur, remove the tray and contact your dentist.
3. Foods and juices high in citric acid can cause sensitivity to the teeth.
4. Some patients have noticed temporary discomfort of the gums, lips, throat, or tongue. Should any of these symptoms persist more than two days or progressively worsen, call your dentist. These side effects will usually subside within 1-3 days after treatment is discontinued.
5. Coffee, tobacco, and other products can re-stain your teeth over time. Should this occur, the teeth can be rewhitened within a few nights with Opalescence.
6. Regular dental checkups and cleanings are important before and after bleaching to maintain a healthy smile.
7. If you have any questions pertaining to this treatment, please ask your dentist.
8. Some old amalgam or "silver" fillings may leave a dark purple color in your bleaching tray; this is normal.
9. Store bleach out of the sun and heat.
10. Discard any unused bleaching gel after treatment is completed.
11. Keep Opalescence out of reach of small children.
12. Do not swallow gel or rinses. Product contains peroxide and may contain fluoride; swallowing large amounts can be harmful.

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EN - Directions: Adults and children 2 years of age or older: Brush teeth thoroughly, preferably after each meal or at least twice a day, or as directed by a dentist or physician. Instruct children under 6 years of age in good brushing and rinsing habits (to minimize swallowing).

Supervise children as necessary until capable of using without supervision. Children under 2 years of age: Consult a dentist or physician.

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© Copyright 2008 Ultradent Products, Inc. All Rights Reserved. U.S. Patent Nos.: 5,098,303; 5,234,342; 5,376,006; 5,409,631; 5,667,386; 5,725,843; 5,746,598; 5,759,037; 5,759,038; 5,770,105; 5,770,182; 5,846,058; 5,851,512; 5,855,870; 5,985,249; 6,036,943; 6,086,855; 6,183,251; 6,306,370; 6,309,625; 6,368,576; and 6,730,316. Australian Patent No.: 654955. European Patent No.: 0522087. Japan Patent Nos.: 3113182 and 2909040. Canada Patent No.: 2078776.